

Stifneck Select

Select and Pedi-Select Extrication Collar

- 980010 Stifneck Select (Qty. 1)
- 980012 Stifneck Select, green (Qty. 1)
- 980020 Stifneck Pedi-Select (Qty. 1)

Manufactured in China for:
LAERDAL MEDICAL AS, Norway
P.O. Box 377, N-4002 Stavanger, Norway
Tel. +47 51 51 17 00,
Fax +47 51 52 35 57

Distributed by:

Canada and Latin America:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel. +1 (416) 298-9600,
Toll free 888/LAERDAL (523-7325)
ou en français (800) 567-9987
Fax +1 (416) 298-8016

USA:
LAERDAL MEDICAL CORPORATION
167 Myers Corners Road, P.O. Box 1840
Wappingers Falls, New York 12590-8840
Tel. (800) 431-1055, +1 (845) 297-7770
Fax (800) 227-1143, +1 (845) 298-4545



Intended for single patient use.



The product is in compliance with the essential requirements of Council Directive 93/42/EEC Medical Devices Directive.

U.S. Patent Nos. 5,795,315; 5,622,529, Other U.S. and International Patents Apply.

© 2005 Laerdal Medical AS. All Rights Reserved.

Printed in China.

C1012 Rev B



Laerdal


helping save lives

www.laerdal.com

Select and Pedi-Select Extrication Collar

1 Measure the patient


Align the head to neutral or 'eyes forward' position unless contraindicated by your protocol.

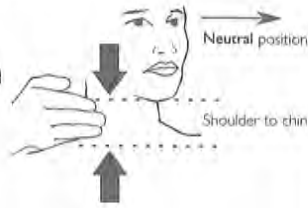

2 Match the collar size to the patient


Select: Choose from 4 adult positions.
Pedi-Select: Choose from 3 children positions.

Same basic method of adjusting for both Collars.

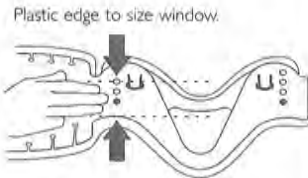


3 Adjust and Lock the adjustable collar

Adjust the chin support to the size selected in step 2.

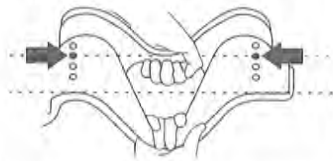

Same sizing method for Select and Pedi-Select

Lock both sides by pressing the two lock tabs.

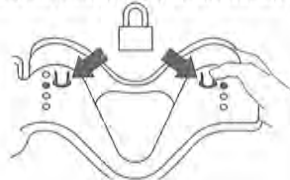

4 Preform the collar



5 Apply the collar while manually maintaining neutral head position



Place the chin support well under the chin. If a different size is needed, remove, re-size, and re-apply the collar.



For a supine patient, slide the rear panel behind the neck before placing the chin support.

Important: Do not adjust the Select/Pedi-Select collar on patient.



Pull the back of the collar snug while holding the front in place, then fasten.

Storage: Do not store collar in folded position. Store flat.

Storage Temperature Range:
-34°C to 52°C
(-30°F to 125°F)

Operating Temperature Range:
-25°C to 43°C
(-13°F to 110°F)

IMPORTANT

All Stifneck products should only be used by persons who have received adequate training. In cases of suspected spinal injury, proper cervical immobilization is only one part of an immobilization system. It is important that the patient be properly immobilized to prevent movement of the spine (per local protocol).

Do not use an improperly sized collar. Too large a collar may hyperextend a patient's cervical spine; too small a collar may not provide appropriate stability. Special sizes of Stifneck collars are available for children and others with small bodies.

WARNING

To insure proper use, please review all material in this DFU.

WARRANTY

Please see the Global Warranty statement for terms and conditions.

www.stifneck.com

Federal law (USA) restricts this device to sale by or on the order of a physician.



intended for single patient use.



The product is in compliance with the essential requirements of Council Directive 93/42/EEC Medical Devices Directive.

Stifneck Select

Select and Pedi-Select Extrication Collar

Directions for Use



- 980010 Stifneck Select
- 980012 Stifneck Select, green
- 980020 Stifneck Pedi-Select



Intended Use

This product is intended to support cervical spine in a neutral position during transportation, in combination with other cervical and full body immobilization devices (CSIDs and FBSIDs).

Important


- In cases of a suspected spinal cord injury, proper neck immobilization is only one part of total immobilization.
- It is imperative that the patient be properly immobilized to prevent any movement of the spinal column.

Warnings

- Stifneck Select Collars should only be used by qualified healthcare providers.
- Improper use may result in injury or permanent disability. Always follow local spinal immobilization protocol.
- Use the correct size collar. An oversized collar may hyperextend a patient's cervical spine. An undersized collar may not provide appropriate stability.
- Do not adjust the Select or Pedi-Select collar on patient.

User Instructions

1. Measure the Patient

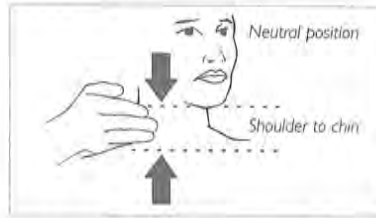


Align the head to neutral or "eyes forward" position unless contraindicated by local protocol.


2. Match the collar size to the patient:

Select: Choose from 4 adult positions.


Pedi-Select: Choose from 3 Paediatric positions.

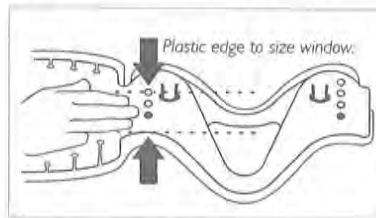


3. Adjust and Lock the adjustable collar

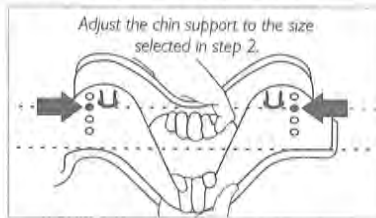

Use the same method for adjusting both Collars.

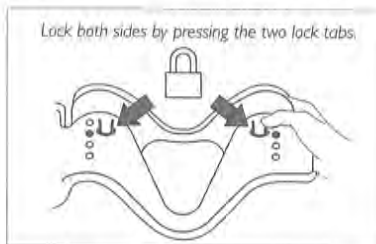

4. Preform the collar

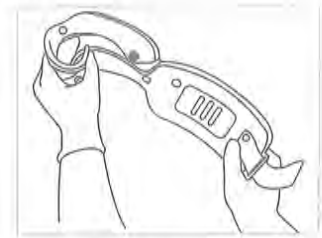

5. Apply the collar while manually maintaining neutral head position



Place the chin support under the chin. If a different size is needed, remove, resize, and re-apply the collar.



Pull the back of the collar snug while holding the front in place, and then fasten.



For a supine patient, slide the rear panel behind the neck before placing the chin support.

Technical Specifications

Storage:
Do not store collar in folded position.
Store in flat position.

Storage Temperature Range:
-34 °C to 52 °C
(-30 °F to 125 °F)

Operating Temperature Range:
-25 °C to 43 °C
(-13 °F to 110 °F)

Manufactured in China for:
LAERDAL MEDICAL AS, Norway
PO. Box 377, N-4002 Stavanger, Norway
Tel. +47 51 51 17 00,
Fax +47 51 52 35 57

Distributed by:
Canada and Latin America:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel. +1 (416) 298-9600,
Toll free 888/LAERDAL (523-7325)
ou en français (800) 567-9987
Fax +1 (416) 298-8016

USA:
LAERDAL MEDICAL CORPORATION
167 Myers Corners Road, P.O. Box 1840
Wappingers Falls, New York 12590-8840
Tel. (800) 431-1055, +1 (845) 297-7770
Fax (800) 227-1143, +1 (845) 298-4545

Australia
Laerdal Pty. Ltd.
8 Stamford Road
Oakleigh VIC 3166.



Laerdal

helping save lives